

# QUICK GUIDE



## RZ153+ SERIES **CARDIO ID+™** RECORDER SETUP

### Programming a New Patient:






- ♥ Insert a Fresh AA Alkaline Battery.
- ♥ Insert the CompactFlash Card into the recorder.  
(The Card will automatically be erased.)
- ♥ Insure that the Patient Cable is attached.
- ♥ Hit any button; the Following screen will appear.
  
- ♥ The "ID" screen will appear in a few seconds.
- ♥ To navigate the "**Function**" button, press the ◀, ▶, ▲, and ▼ arrows on the key to move the cursor to the correct location and the ↵ (Enter) button to select that item.
- ♥ Use the ◀ and ▶ arrows to go to the proper numbers for the Patient ID and ↵ (Enter) to select.
- ♥ Use the ▼ arrow and go to the second line of text to go to "**Back**" (to change a number) or to "**Done**" to finish entering the Patient ID.

**NOTE:** You cannot move on without a Patient ID!

- ♥ Push the ▶ arrow to change the active Tab (screen) to "**CH1**" to view the ECG. (This is where you check the Hookup and change the gain. The Gain will be set for **ALL 3 Channels**.)
- ♥ Moving to the "**CH1**" Tab will display the Patient's ECG for Channel 1. Likewise CH2 and CH3 will display ECG Channels 2 and 3.
- ♥ The ▲ and ▼ arrows will increase or decrease the gain for all channels.
  
- ♥ Push the ▶ arrow to change the active Tab (screen) to "**Start**".
- ♥ Pressing the ↵ (Enter) button at the Start Tab will begin the Holter recording.
- ♥ During recording the **Cardio ID+** displays the current time, time remaining to the end of test and a battery-life indicator.

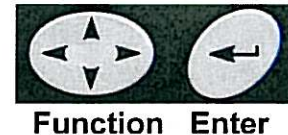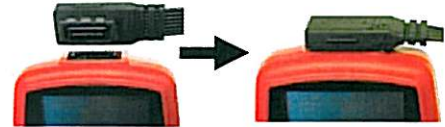

### Patient ID

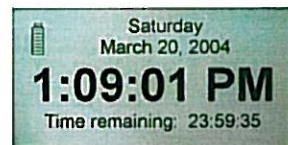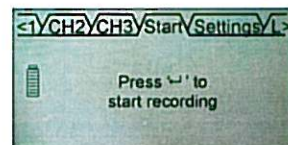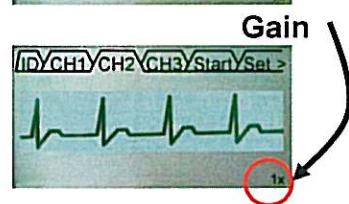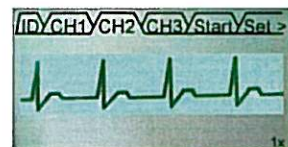
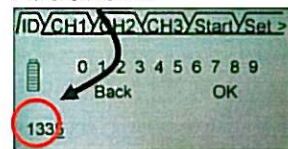


**SEE OTHER SIDE ↓**

**ROZINN**



## Changing the Cardio ID+ Settings:

- ♥ To change the Recorder Settings, use the ► arrow to change the active Tab (screen) to “Settings”.
- ♥ The ◀, ▶, ▲, and ▼ arrows move the cursor to the correct location and the ↵ (Enter) button selects that item.
- ♥ The “Time / Date” are set in the same way.

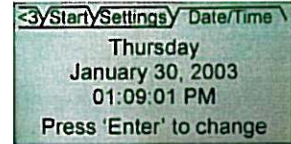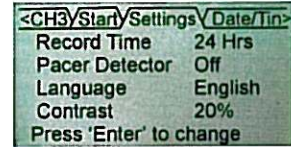

## Early Out Procedure:

- ♥ The Cardio ID+ supports an “Early-Out” feature that allows you to stop a recording before the end of the selected recording time (24 or 48 Hours).
- ♥ To initiate an Early-Out hold the ◀ on the Function button and the ↵ (Enter) button simultaneously.
- ♥ You may end the test or view the Patient’s ECG.

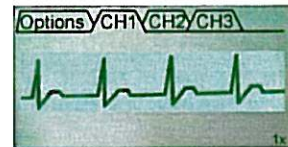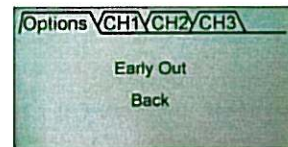

## WARNING:

- ♥ Failure to follow the procedure above may result in the loss of **all** recorded Patient ECG (Holter) data.
- ♥ The default for Pacemaker Detection is OFF. You must turn it on for every Pacemaker Patient.
- ♥ The default for Record Time is 24 Hours. You must set it to 48 Hours for every 2-Day Test.

## Patient Electrode Hookup:

