


## H3+™ DIGITAL HOLTER RECORDER

### To Begin a Session

1. Attach the patient cable to the H3+.
2. Remove the battery door of the H3+, insert a new AAA battery in the compartment and close the battery door.

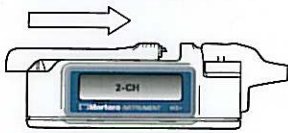

3. Press the Enter button to move through the menu displays.



- Verify the current time setting.



- Verify the ID number.




- Verify the ECG signal quality.



**Note:** Waveform is shown at 4 mm/mV gain to allow full representation of ECG.


4. To begin recording, press and hold the Enter

button for a period of 3 seconds. An R indicates Recording Mode.

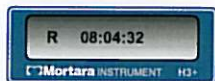

### Entering Optional Diary Events

To mark an event, press the Enter button. A ↓ indicates that a new event cannot be entered while the ↓ is displayed.

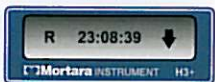


### Ending a Recording

When the recording period has ended, the ID is displayed in reversed color indicating that the H3+ memory is full.



1. Remove the battery door.
2. Remove the battery and dispose of the battery properly.
3. Replace the battery door.
4. Remove the patient cable from the recorder.
5. H3+ recorded data is ready to be acquired at the H-Scribe system through connection of the USB cable.

## 3-CHANNEL Holter electrode placement

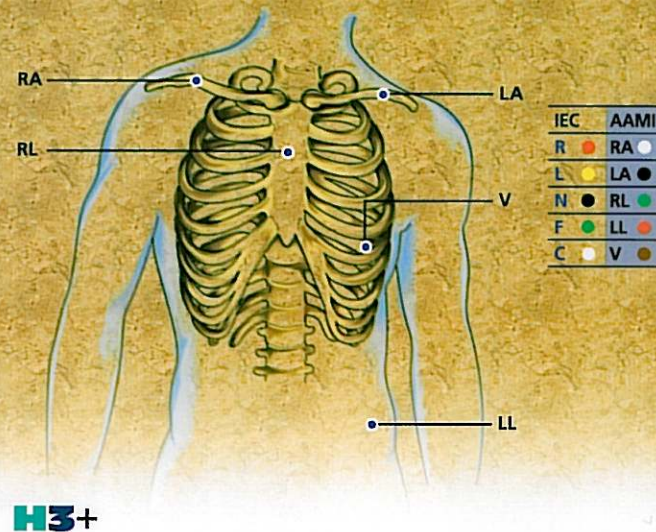

### LEAD PLACEMENT

- **RA** – Right clavicle as shown.
- **LA** – Left clavicle as shown.
- **RL** – Reference or ground lead, should be placed to maximize patient comfort.
- **LL** – Lower left side of the rib cage or body.
- **V** – Precordial exploring lead.

The V lead can be positioned according to clinician preference.

Mortara Instrument, Inc.  
7865 North 86th Street  
Milwaukee, WI 53224  
Tel: 414.354.1600  
Tel: 800.231.7437  
Service: 888.MORTARA  
Fax: 414.354.4760

Mortara Rangoni Europe, Srl  
(European Headquarters, Italy)  
Via Cimara 103/105  
40033 Casalecchio di Reno (BO)  
Tel: +39.051.298.7811  
Fax: +39.051.613.3582

Mortara Instrument GmbH  
(Germany)  
Kaninenberghöhe 50  
45136 Essen  
Tel: +49.201.268311  
Fax: +49.201.268313

Mortara Instrument B.V.  
(The Netherlands)  
H. Dunantplein 6  
3731 CL De Bilt  
Postbus 131  
3720 AC Bilthoven  
Tel: +31.30.2205050  
Fax: +31.30.2201531


**Mortara INSTRUMENT**  
The World Leader in ECG Technology

<http://www.mortara.com>  
ISO 13485 Certified

**Mortara INSTRUMENT**  
The World Leader in ECG Technology



# 2-CHANNEL Holter electrode placement

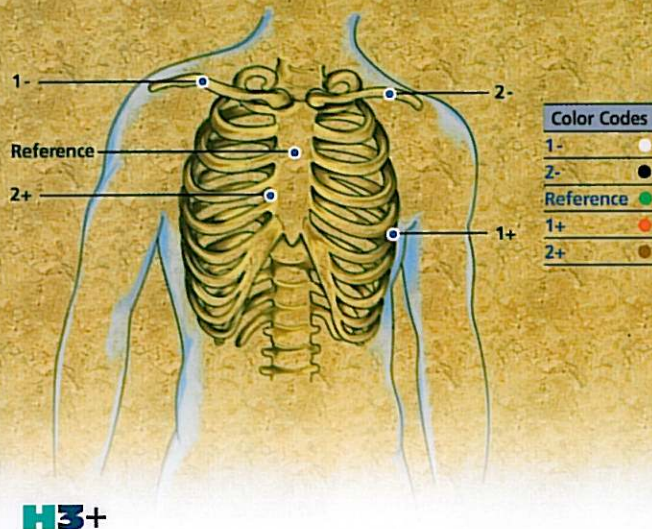

**H3+**

## LEAD PLACEMENT

- **1- -**  
Channel 1 negative lead.
- **1+ -**  
Channel 1 positive lead.
- **2- -**  
Channel 2 negative lead.
- **2+ -**  
Channel 2 positive lead.
- **Reference Lead -**

The leads can be positioned according to clinician preference.

**Mortara INSTRUMENT**  
The World Leader in ECG Technology

Mortara Instrument, Inc.  
7865 North 86th Street  
Milwaukee, WI 53224  
Tel: 414.354.1600  
Tel: 800.231.7437  
Service: 888.MORTARA  
Fax: 414.354.4760

Mortara Rangoni Europe, Srl  
(European Headquarters, Italy)  
Via Cimara 103/105  
40033 Casalecchio di Reno (BO)  
Tel: +39.051.298.7811  
Fax: +39.051.613.3582


Mortara Instrument GmbH  
(Germany)  
Kaninenberghöhe 50  
45136 Essen  
Tel: +49.201.268311  
Fax: +49.201.268313

Mortara Instrument B.V.  
(The Netherlands)  
H. Dunantplein 6  
3731 CL De Bilt  
Postbus 131  
3720 AC Bilthoven  
Tel: +31.30.2205050  
Fax: +31.30.2201531

## H3+™ DIGITAL HOLTER RECORDER

### To Begin a Session


1. Attach the patient cable to the H3+.
2. Remove the battery door of the H3+, insert a new AAA battery in the compartment and close the battery door.

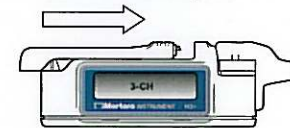


3. Press the Enter button to move through the menu displays.



- Verify the current time setting.

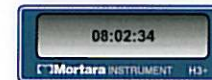

- Verify the ID number.

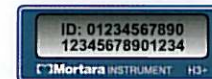


- Verify the ECG signal quality.



**Note:** Waveform is shown at 4 mm/mV gain to allow full representation of ECG.


4. To begin recording, press and hold the Enter

button for a period of 3 seconds. An R indicates Recording Mode.

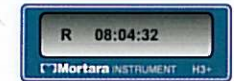

### Entering Optional Diary Events

To mark an event, press the Enter button. An ↓ indicates that a new event cannot be entered while the ↓ is displayed.

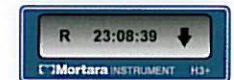

### Ending a Recording

When the recording period has ended, the ID is displayed in reversed color indicating that the H3+ memory is full.



1. Remove the battery door.
2. Remove the battery and dispose of the battery properly.
3. Replace the battery door.
4. Remove the patient cable from the recorder.
5. H3+ recorded data is ready to be acquired at the H-Script system through connection of the USB cable.